quick2go[®]

Türschwellerampe

Door Threshold Ramp

Passage de Seuil

Drempelhulp

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 User manual.....Page 06 - 09

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Türschwellenrampe quick2go®

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Door Threshold Ramp *quick2go*[®]

Thank you very much for purchasing our door threshold ramp and for your trust in ROLKO products. Before installing and using this product, please carefully read the entire user manual. Check that all parts are included and follow all of the safety instructions.



Application / Purpose

The threshold ramp quick2go allows the overcoming of door thresholds, steps and other obstacles. Suitable for wheelchairs, electric chairs, quads, etc. with a weight up to 250 kg. Based on the height and nature of the obstacle you receive the appropriate number of wedges, blocks and clips for the ramp.

Delivery Content

The delivery content varies according to ramp model:

Article number	Obstacle height	Number of ramp parts			
		Wedge	Block	VB1	VB2
Rampe-TS-11	11 mm	1	0	0	0
Rampe-TS-22	22 mm	2	1	2	2
Rampe-TS-33	33 mm	3	3	4	6
Rampe-TS-44	44 mm	4	6	6	12
Rampe-TS-55	55 mm	5	10	8	20


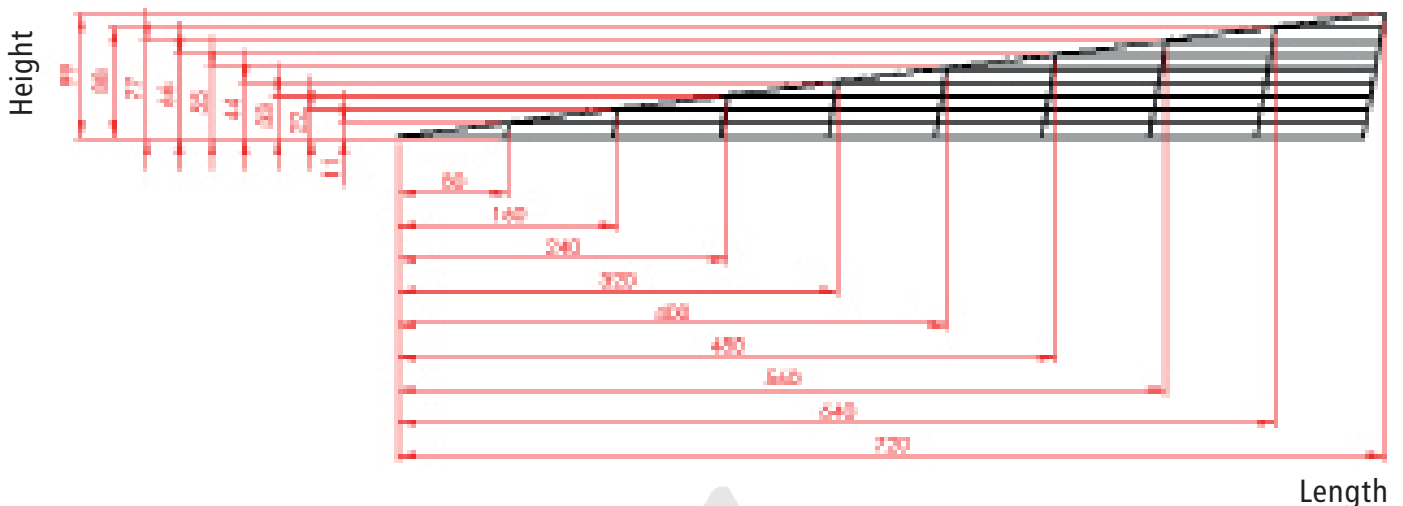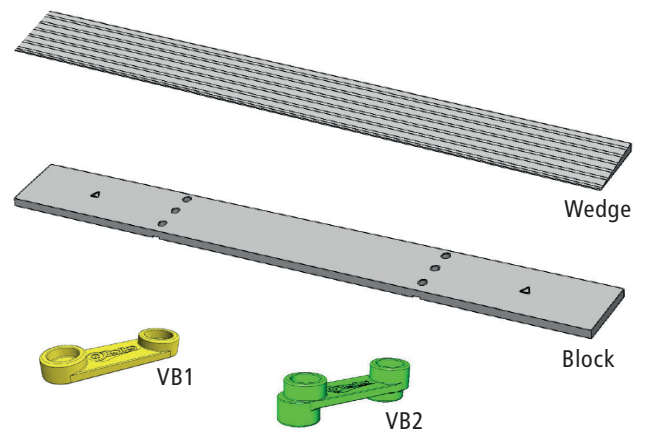
Safety Instructions

- The ramps is to be used only at a moderate speed, please avoid sudden acceleration or deceleration.
- Be very careful while using the ramp during strong weather conditions like snow, frost or wetness.
- The non-slip material of underside of the ramp has to rest evenly.
- The use of ramps with damages is not allowed! Spare parts can be ordered through specialist retailers.



Assembly/Use

The threshold ramp quick2go can be modularly build in several layers, according to the obstacle height. Based on the following pictures you can see how the ramp parts are mounted:


1. For the first layer lay out the required number of rubber parts. The arrows on the upper side of the rubber blocks show the obstacle direction (see fig. A). Connect all elements from below with the clip connectors VB1 (see fig. B).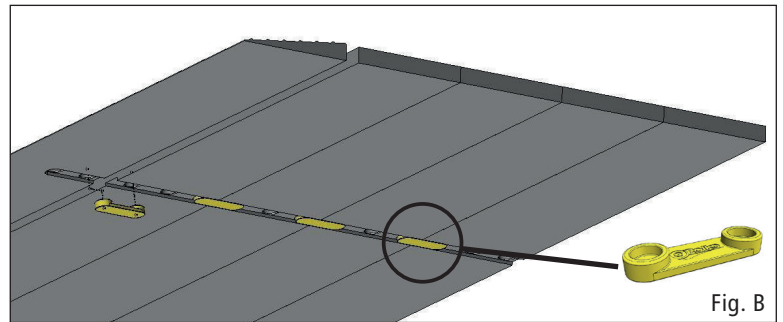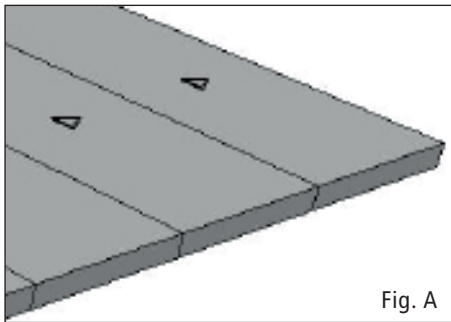



2. Insert the clip connectors VB2 from above, observe the correct orientation (fig. C + D)! Connect all rubber parts lying together and additionally insert in the last two holes of the rubber part a clip connector VB2.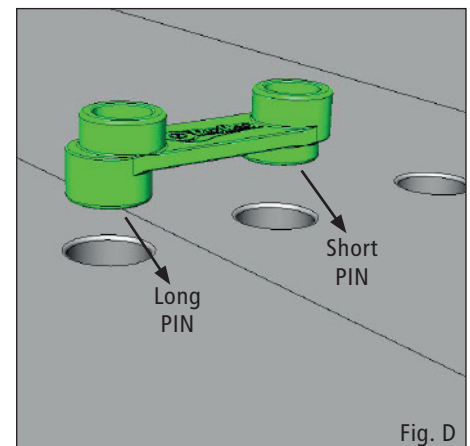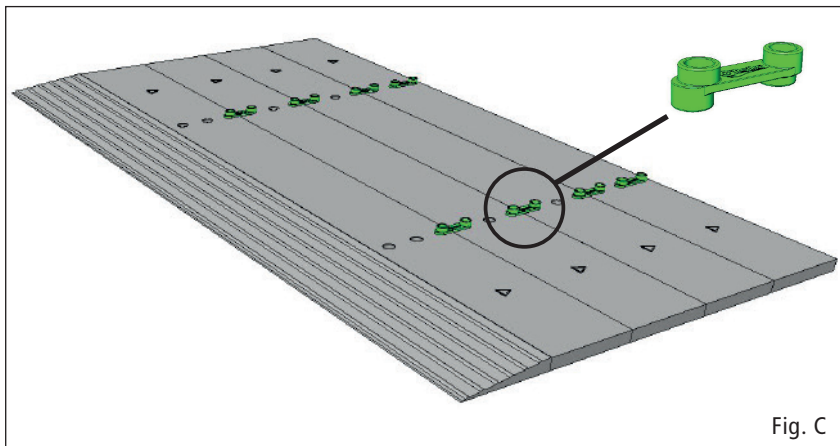

3. For the second ramp layer insert another layer of rubber parts on the already attached clip connectors. Continue mounting the rampe alternating with rubber parts and clip connectors as described in step 2 (see fig. E).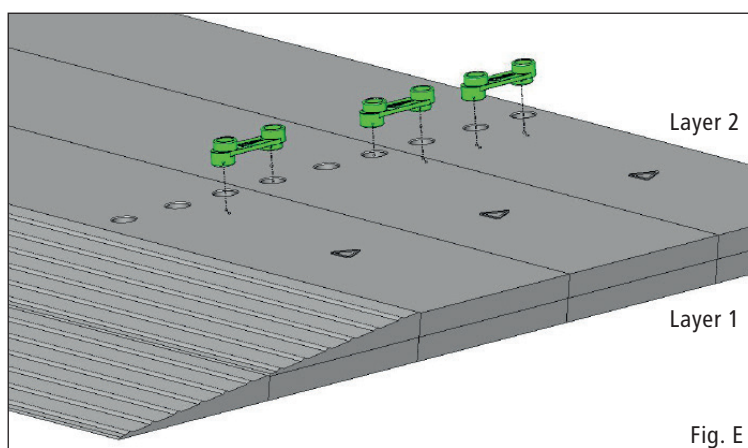



4. On the topmost rubber block connect 2 clip connectors VB2 in reversed position (see fig. F + G).

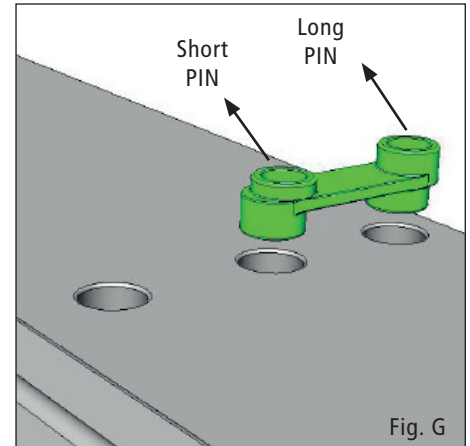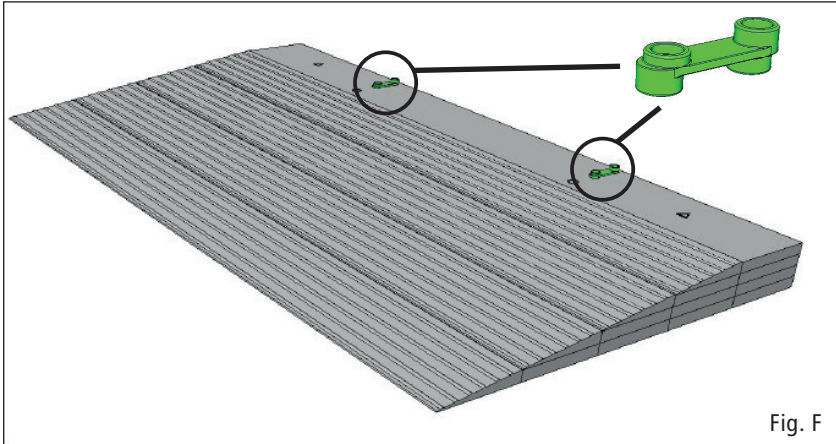5. After that place the top and last rubber wedge on the ramp and press it firmly. The door threshold ramp quick2go is now ready for use (see fig. H).

6. OPTIONAL: To avoid the ramp from falling apart during transportation, the clip connectors VB1 and VB2 can be glued to the ramp parts. For the ramps with the article numbers -TS33, -TS44 and TS55 a suitable glue is already included in the scope of delivery (see fig. I + J).

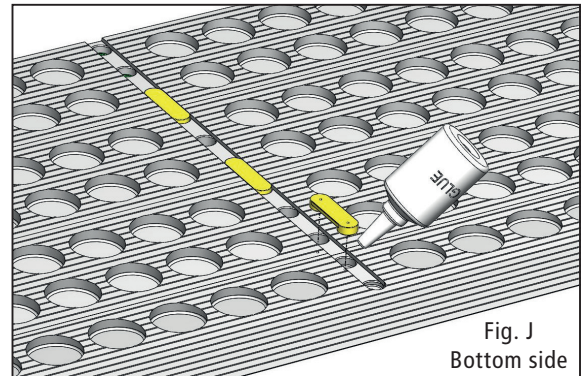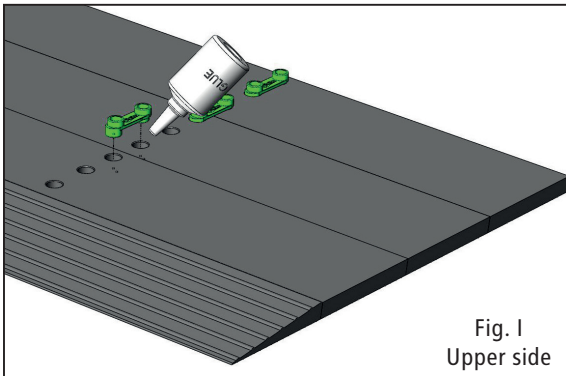As an accessory for medical products of class 1, this product has a CE mark in accordance with regulation (EU) 2017/745 (MDR) and thus complies with all German and European safety requirements, that apply to the product. The product fulfils the corresponding sections of standard specification DIN EN 12182. The EU declaration of conformity is available for download at instructions.rolko.com.

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Reprint, also in extracts, only with prior permission by the editor. All rights reserved.

The delivered product may differ from the illustrations in this manual. All technical data are characteristic and subject to construction and manufacturing conditioned tolerances.

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© Rolko Kohlgrüber GmbH, Industriestr. 14 - 16, 33829 Borgholzhausen, Duitsland